made
in
Germany

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made
in
Germany

How to use the Cardio First Angel

Before using the Cardio First Angel in an emergency, you should always call the ambulance (112) first. Then provide first aid. The easy to understand pictograms printed on the device guide you through the resuscitation process, step-by-step. Keep calm. As long as you follow the instructions you can be assured, that the patient is helped properly and effectively. Continue CPR until the ambulance arrives.

PRAXISDIENST
Medical Supplies since 1953

[Order here!](#)


Simply place the Cardio First Angel in the center of the chest. The shape of the device helps you with the correct position.

Kneel next to the patient and perform CPR using the Cardio First Angel. Use your shoulders strength.

The click sound indicates how hard you have to press and aids you with the frequency.

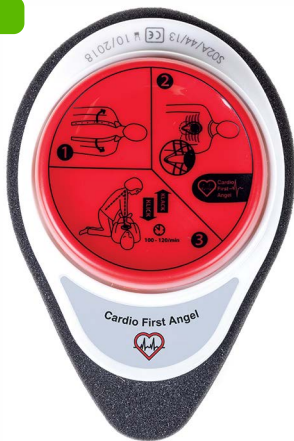made
in
Germany

CPR-Feedback System

Easy to use

The Cardio First Angel is easy and intuitively to use.
The shape of the device helps you finding the correct position on the patients chest.
The click sound indicates the correct compression depth and the correct

Mechanical

The Cardio First Angel works mechanically.
No battery or power source required. That makes the Cardio First Angel available and ready everywhere and anytime.
Your reliable first aid helper.

Harmless material

The Cardio First Angel is entirely made out of harmless biocompatible materials.
The Cardio First Angel is CE certified.

Compact design

Thanks to its small form factor the Cardio First Angel is quickly available and fits almost everywhere. With only 130 grams it would even fit in a small purse.

No skin contact

Are you afraid of touching an injured person?
The Cardio First Angel created a "buffer" between you and the victim.
Many first aiders are more willing to resuscitate without direct skin contact.