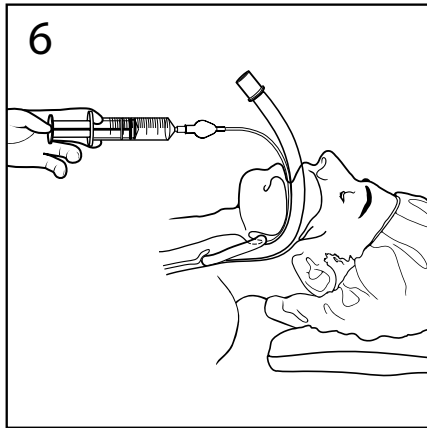
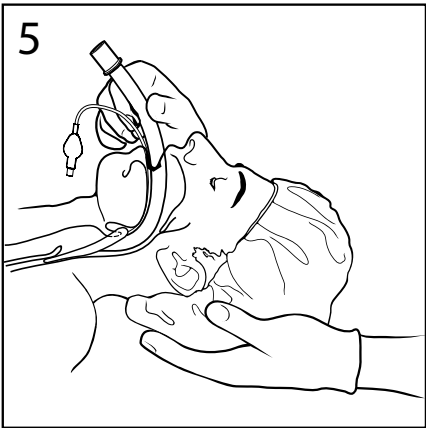
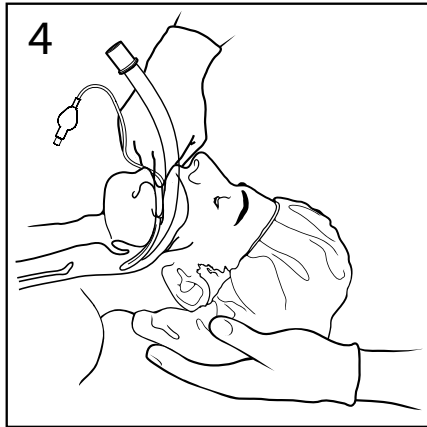
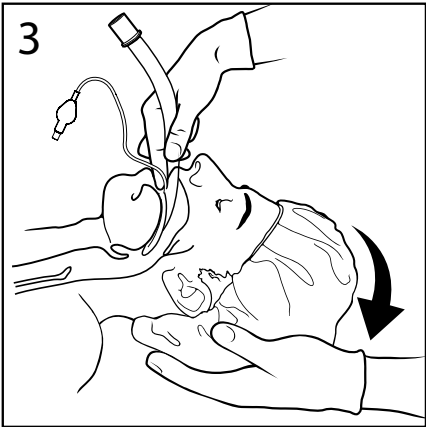
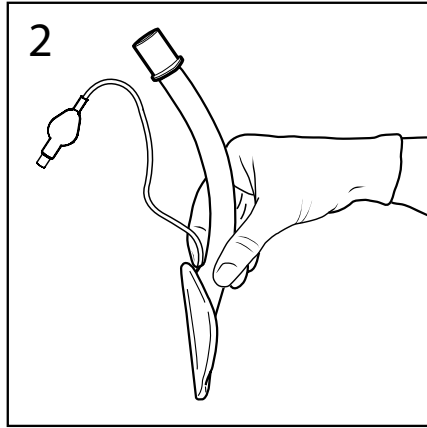
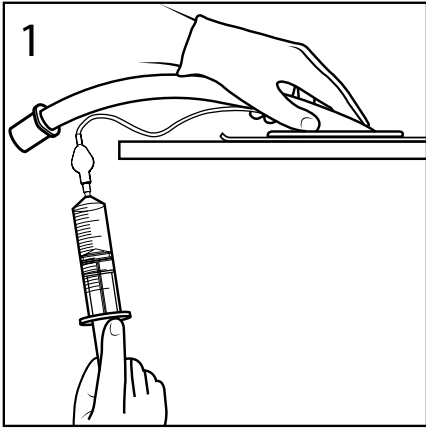
Instruction for use

Ambu[®] AuraStraight[™]
Single Use Laryngeal Mask - Sterile

For use by medical professionals trained in airway management only

Content

Page

English	4
България	5
Česky	6
Dansk.....	7
Deutsch	8
Ελληνικά	9
Español.....	10
Eesti.....	11
Suomi.....	12
Français.....	13
Hrvatski.....	14
Magyar	15
Italiano.....	16
日本語	17
Lietuviškai.....	18
Latviski.....	19
Nederlands	20
Norsk.....	21
Polski.....	22
Português.....	23
Romania	24
Русский	25
Slovenčina.....	26
Slovenščina.....	27
Svenska	28
Türkçe	29
中文	30

1. Intended use

The Ambu AuraStraight is intended for use as an alternative to a facemask for achieving and maintaining control of the airway during routine and emergency anaesthetic procedures in fasted patients.

Intended users: "Professionals trained in airway management".

Have a spare Ambu AuraStraight ready and prepared for immediate use.

Contents sterile unless packaging is damaged or opened.

WARNINGS

The user should be familiar with the following warnings prior to use of the AuraStraight.

- Do not use the AuraStraight before precheck: Visual inspection, air leak test of the device. Failure of any test indicates that the device should not be used.
- All Ambu AuraStraight masks must, prior to use, be investigated and inspected for any potential foreign body. The cuff-protector only serves as protecting the Ambu AuraStraight during storage and transportation and must be removed prior to use.
- Do not use the Ambu AuraStraight if there are any bulges on the cuff or if there are any signs of leakage.
- To avoid trauma, do not use excessive force.
- The Ambu AuraStraight is flammable in the presence of lasers and electrocautery equipment.
- For single use only. Use on other patients can cause cross contamination. Do not soak, rinse, or sterilize this device as these procedures may leave harmful residues or cause malfunction of the device. The design and material used are not compatible with conventional cleaning and sterilization procedures.
- The cuff volume or pressure may change in the presence of nitrous oxide, oxygen, or other medical gases.
- The AuraStraight does not protect the trachea or lungs from the risk of aspiration.

CAUTION

- US federal law restricts this device to be sold to or on the order of a physician.
- Ensure that the device is not in any way damaged before use.
- The patency of the AuraStraight should be reconfirmed after any change in the patient's head or neck position.

For further product information see www.ambu.com

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Ambu[®] is registered trademark of Ambu A/S, Denmark.
Ambu A/S is certified according to ISO 13485.
Ambu A/S