CLIPSTOR

DeloX

#18463

- DE Bedienungsanweisung
- FR Mode d'emploi
- EN Instructions for use
- IT Istruzioni per l'uso
- NL Gebruiksaanwijzing
- ES Instrucciones de uso
- FI Käyttöohje
- SE Bruksanvisning
- PL Instrukcja obsługi

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1. Sicherheit

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EN Operating instructions

1. Safety

Please read the instructions for use carefully before operating the clippers, especially the „Safety“ section.

This will help you avoid possible hazards and protect against damage to the device due to misuse.

Keep the instructions for use handy for future reference. If the device is passed on to a new owner, please also hand over these instructions for use.

1.1 Explanation of terms and symbols

The following terms are used in these instructions for use:

DANGER

If disregarded, can lead to serious injuries or death

WARNING

If disregarded, can lead to serious injuries (or death)

Note

Information that should be observed when handling the device.

Danger of an electric shock! The device may never be charged or operated while in the bath, shower, over a filled washbasin / other container.

1.2 Intended use

- the clippers are used to trim hair/fur
- the device is designed for indoor use only. It should not be operated outdoors (garden, terrace, etc.).
- if the device is used in a manner that does not comply with the intended use and/or if modifications are made to the device, then this voids the manufacturer's warranty and liabilities

DANGER

1.3 Important danger warnings

- keep the device, charging tray and connecting cable away from moisture, water and other liquids, and do not operate it with wet hands
- do not immerse the device, charging tray and connecting cable under water or rinse off under running water
- if the device should fall into water, it may not be used again afterwards
- the integration of a residual-current-operated protective device, with a nominal tripping current of less than 30 mA, in the building installation provides additional protection. We recommend that you have your electrician install residual current protection for your power circuits.
- the device must be disconnected from the power supply when the battery is charged or after at least 8 hours of pure charging time
- if the battery does not fully charge after 8 hours, do not operate the device and contact the manufacturer

WARNING

The clipping head can become hot after extended use!

DANGER

The clipper may only be operated with a power pack that meets the specifications on the nameplate. The device may only be operated with a power pack that delivers the SELV output voltage specified on the device's markings.

1.4 General safety instructions

DANGER

Never operate the device if:

- the device, charging tray or connecting cable is damaged
- the clipping head is damaged or chipped because this can injure the skin
- the device is dropped, shows visible damage or its functionality is impaired
- the device is damaged so that electrical components are exposed

In each case, the device must be disconnected from the mains immediately!

This device can be used by children 8 years of age and older and by individuals with impaired physical, sensory or mental abilities or those with insufficient experience and knowledge to do so, if they are being supervised or have been trained on the safe use of the device and understand the underlying hazards. Children may not play with the device. Cleaning and user maintenance may not be performed by children without supervision.

- children should be supervised at all times to ensure that they do not play with the device. Make sure that children do not injure themselves on the clipping head. Keep the device out of the reach of children.
- also keep the packaging material, e.g. films, away from children
- lay the connecting cable in such a way so that no one can trip over or step on it
- protect the connecting cable from damage. Never pull on the connecting cable. Always remove the cable from the power outlet by grasping the power pack plug. Keep the cable away from hot surfaces. Check the connecting cable for damage on a regular basis.
- never expose the device to temperatures under 0 °C or over +40 °C for long periods of time. Avoid direct sunlight.
- repairs may only be performed by authorised personnel in order to avoid a hazard
- disconnect the device from the power supply for maintenance and parts replacement
- the clipping head may not be taken apart

1.5 Safety during operation

DANGER

Never touch or reach into the clipping head when the device is switched on.

The clipping head must be cleaned on a regular basis and oiled at the corresponding locations to ensure problem-free operation.

The device must be switched off when:

- you put the device down, e.g. to charge it or when you interrupt the cutting process for a brief period
- you attach or remove the comb attachment
- you clean or oil the device

Only use the clipper together with original accessories (charging tray, comb attachment, etc.).

Never use sharp objects (e.g. a tail comb) to remove dust or foreign objects from the interior of the device.

2. Charging

- when the device is delivered, it is not yet ready for operation
- before connecting the clipper to an outlet, make sure that the mains voltage matches the voltage listed on the nameplate of the power pack
- in order to maintain the charge capacity of the battery, we recommend that you occasionally discharge the battery by operating the device until the motor stops
- please charge the device for a full 8 hours before first use so that the included rechargeable battery is activated
- please do not charge the device in temperatures under 0 °C or higher than 40 °C
- please do not charge the device while exposed to direct sunlight or the heat of a radiator
- the standard charge time is 300 minutes. Avoid extended charge times to prevent damage to the battery.
- when the device is fully charged, it can be used for longer than 300 minutes. During charging, the following symbol lights up on the LED display:

- the clippers can be connected directly to the power supply using the power pack or charged using the charging tray and power pack

3. Scope of delivery

- 1 Clipper
- 2 Comb (3 mm)
- 3 Comb (6 mm)
- 4 Comb (9 mm)
- 5 Comb (12 mm)
- 6 Cleaning brush
- 7 Oil
- 8 Power pack
- 9 Charging tray
- 10 LED light
- 11 On/Off switch / Cutting speed
- 12 LED display

4. Operation, cleaning and care

Switching on and off:

Press the On/Off switch to switch off the clippers (fig. 1). You can press it again to increase the cutting speed and then switch it off. You must press and hold the On/Off switch for a min. of 3 sec. to turn off the LED light. If the On/Off switch is pressed again for 3 sec., the LED light turns off again.

The LED display shows the remaining shearing time in minutes (fig. 2). When the device is switched off, the LED display shows the shearing time in seconds.

This clipper can be used via its cable as well as without its cable.

Connect the power pack to the device socket located on the bottom of the device. Then connect the power pack to the power (fig. 3). The charging tray can also be used with the power pack (fig. 4).

Cutting height adjustment:

The clippers have a 5-stage, cutting height adjustment (0.8 - 2.0 mm). The cutting height can be changed by turning the setting wheel (fig. 5).

Using the comb accessory:

Make sure that the clipper is switched off when installing and removing the comb. Four combs are available (3, 6, 9, 12 mm) for different cutting lengths. Place the comb onto the clipping head, making sure that it engages on the side (fig. 6).

Clean and oil the device on a regular basis to ensure the problem-free function and full functional performance of the clippers.

Cleaning the clipping head:

The clipping head must be cleaned when the cleaning symbol lights up.

Switch the device off. Disconnect the device from the power supply. Remove the comb attachment and press in the centre of the shearing blades to remove them (fig. 7). Remove the hair with the cleaning brush.

Place the clipping head back onto the clippers as shown in fig. 8. Make sure that the locking mechanism is positioned properly and press the clipping head onto the device until it engages.

Oiling the clipping head:

The clipping head must be oiled when the following symbol lights up in order to maintain the good cutting performance of the clipping head:

Remove the comb attachment. Use the brush to clean the cutting edge, removing the hair between the blades. Oil the locations where the upper and lower part of the clipping head touch. The device must be oiled immediately when the shearing performance diminishes noticeably or the temperature of the clipping head seems too high. The device must be oiled immediately when the shearing performance diminishes noticeably or the temperature of the clipping head seems too high. The clipping head should be oiled once per day when the clippers are not in use.

Replacing the clipping head:

The clipping head is a wear part. Despite good care, the cutting performance will diminish. At this point, the clipping head should be replaced. The clipping head can be ordered individually as a spare part (ref. no. 18477). When replacing the clipping head, you can remove the residue hair within the clippers using the brush. Press in the centre of the shearing blades to remove them (fig. 7). Place the new clipping head back onto the clippers as shown in fig. 8. Make sure that the locking mechanism is positioned properly and press the clipping head onto the device until it engages.

Cleaning the device:

Wipe off your device occasionally using a moist cloth, then rub it dry.

5. Troubleshooting

What to do, when...

... the device cannot be switched on?

- is the battery empty? Connect the power pack and charge the clippers
- is the clipping head dry? The clipping head must be oiled

... the clipping head does not operate smoothly?

- the clipping head must be oiled
- the top and bottom blades can become stuck when not in use for longer periods of time

... the charging display does not light up when charging?

- make sure that the power pack is connected properly

... the device switches off by itself?

- the battery is empty. Charge the clippers

... the battery can no longer be charged?

- the battery or the power pack is defective. The power pack can be re-ordered. If the battery is defective, then the device must be disposed of properly.

6. Technical data

	Clipper	Power pack
Art. No.	18463	18468
Motor speed	4000/5000/6000 rpm	-
Charging time	300 min	-
Shearing time	300 min	-
Weight	242 g	69 g
Input voltage	5V DC	100-240 V
Frequency	-	50/60 Hz
Battery capacity	2200 mAh	-
Current consumption	max. 2000 mA	0.2 A
Ambient temperature charging process/use	0°C - 40 °C	0°C - 40 °C
Protection class	III	II

Electronic scrap

Disposing of this device after its service life is the responsibility of the operator. Please consult the valid national regulations. The device must not be disposed of in household waste. In accordance with the stipulations of the EU Directive on the Disposal of Electrical and Electronic Devices, the device can be disposed of free of charge at the local waste collection or recycling centre. Alternatively, it can be returned to retailers who offer a collection service. The proper disposal helps to ensure environmental protection and prevents any adverse effects on human health and the environment.

CE-symbol / CE-conformity declaration

Albert KERBL GmbH hereby declares that the product / device described in these instructions complies with the fundamental requirements and other relevant stipulations and regulations. The CE mark confirms compliance with the Directives of the European Union.

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Albert Kerbl GmbH
Kerbl Austria Handels GmbH