STAY  
SAFE!

GCL  
**+Vision one**  
**S/M SIZE 250s**

SET CONSISTING OF A  
REUSABLE FRAME  
AND 5 VISORS.

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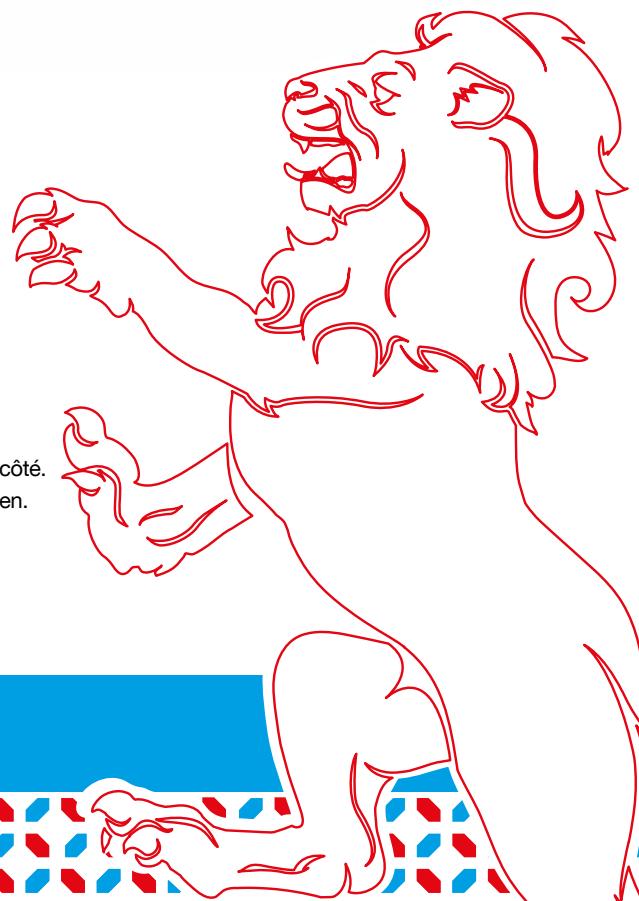
**ENGLISH – CAREFULLY READ THESE USER INSTRUCTIONS BEFORE USING THIS VISOR** These products are classed as Personal Protective Equipment (PPE) by the European PPE Regulation (EU) 2016/425 and have been shown to comply with this Regulation through limited testing against the Harmonised European Standard EN 166:2001 (see below). This product is designed to provide some protection against contact with airborne droplets that may constitute a biological hazard. However, always remember that no item of PPE can provide full protection and care must always be taken while carrying out the risk-related activity. Never use this visor for protection against other hazards for which the product is not design for or tested against. **This product is certified for “use against COVID-19 virus threat and not for general industrial applications”** requiring a specific mechanical performance. **How to assemble the product:** Hold the front plastic band and insert the 2 central pins into the central holes of transparent face shield. Then force the lateral part of the front plastic band straightening it; insert the lateral pin inside the transparent shield. Do the same on the opposite side with the 4th pin. Fix the (black) elastic band on the 2 sides of the front plastic band. Try the face shields to check if it is tight enough; if not, re-adjust the elastic band using a different hole (tighter or looser). **Remove the protective layer on both sides.** **How to wear:** The forehead plastic band should sit approximately 1.5-2.5 cm (1/2 - 1 inch) above the eyebrows, with the bottom of the shield sitting below chin level. **How to clean:** Rinse with warm soapy water; do not use solvent based cleaning agents. **PERFORMANCE AND LIMITATIONS OF USE** – These products have been tested in accordance with EN 166: 2001 Clauses: 6 | Design; 7.1.1 | Field of vision; 7.1.2.1 | Refractive powers; 7.1.2.2 | Transmittance; 7.1.2.3 | Diffusion of light; 7.1.3 | Quality of materials; 7.1.5.1 | Stability at elevated temperature; 7.1.7 | Resistance to ignition; 7.2.4 | Droplets and splashes of liquids **FITTING AND SIZING** – Only wear products of a suitable size that are correctly adjusted. Products which are too loose may not stay in place and if too tight will be uncomfortable. The size of these products is marked on a label attached with the assembly kit and they are available in the following sizes: S/M fitting with head size from 54 cm to 59 cm - M/L fitting with head size from 58 cm to 64 cm. **Additional details:** Before use Try the face shields to check if it is tight enough; if not, re-adjust the elastic band using a different hole (tighter or looser). Remove the protective layer on both sides. Wear the face shield in a way that the forehead plastic band should sit approximately 1.5-2.5 cm (1/2 - 1 inch) above the eyebrows, with the bottom of the shield sitting below chin level. **COMPATIBILITY** – To optimise protection, it will be necessary to use these products with suitable gloves/gowns/masks. In this case, before carrying out the risk-related activity, consult your supplier or supervisor to ensure that all your protective products are compatible and suitable for your application. **Warning -** Parts of the visor that come into contact with your skin may lead to allergic reactions. In this case please discontinue use and seek medical advice. **STORAGE AND TRANSPORT** – When not in use, store the visor in a well-ventilated dark place away from sunlight and extremes of temperature. Never place heavy items on top of it. If possible, avoid excessive folding and preferably store it hanging vertically. If the product is wet, allow it to dry fully before placing it into storage. **OBSCURENCE** – This visor has a shelf life of 2 years. Service life will depend on usage conditions but the visor should be disposed of and replaced if badly scratched or damaged in some way that prevents safe use. **REPAIR** – If the visor becomes damaged, it will NOT provide the optimum level of protection, and therefore should be immediately either replaced or repaired. Never use the damaged product. Repair of any part of the product is not permitted. If in doubt always consult the manufacturer for replacement parts. **CLEANING** – Only use the following recommended cleaning treatment - Rinse with warm soapy water **NEVER** use solvent based cleaning agents. **Declaration of conformity** (<https://gcltechnologies.lu/faceshields>)

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